

AMS Henry Finger Sweep™




HENRY FINGER SWEEP™

Designed for use with the AMS 700MS™ Series.

INSTRUCTIONS:


- ▲ Patient prepped, draped, and catheterized
- ▲ Incision below penal-scrotal junction
- ▲ Sweep distal and down proximally with opposing fingers
- ▲ Bluntly dissect the scrotal fat off both sides of the urethra
- ▲ See instructional CD for surgical technique

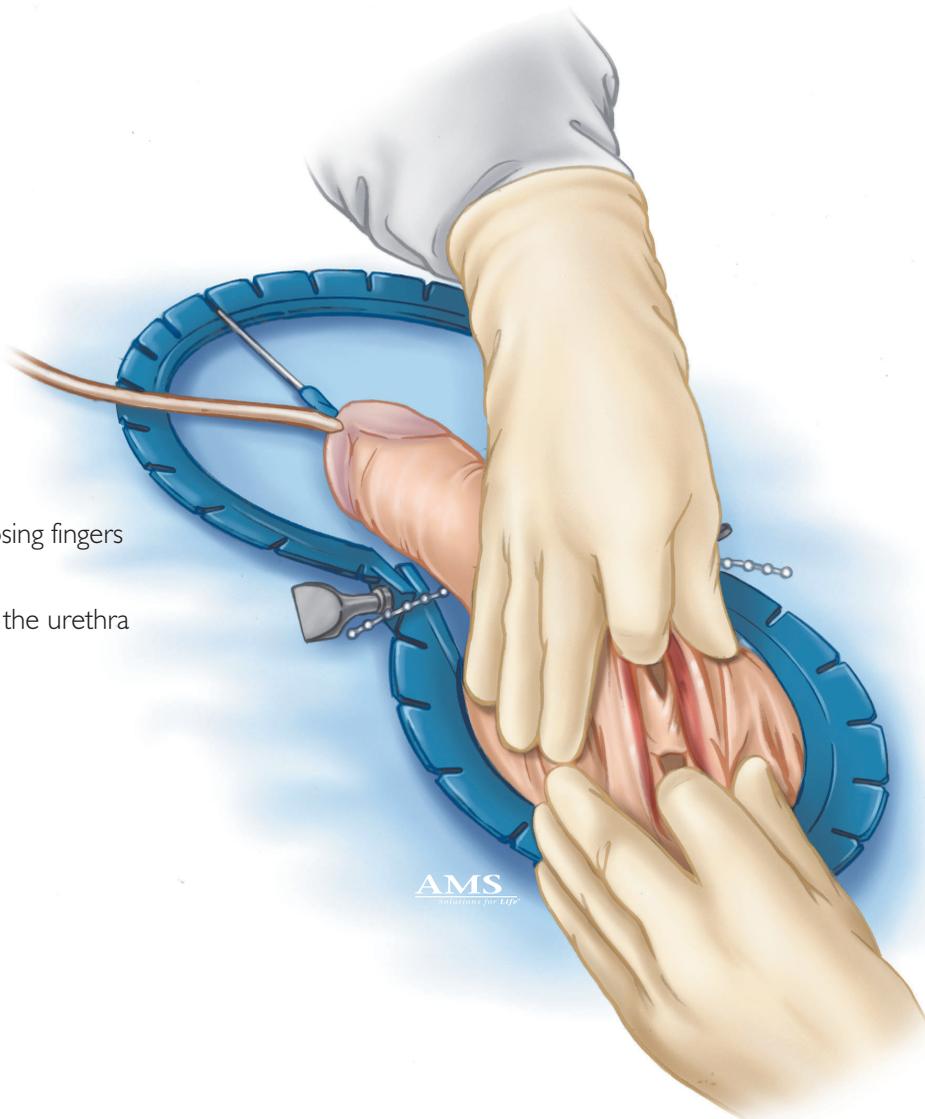

Disclaimer: This surgical footage is for educational and promotional purposes. Stated results are unique to Dr. Henry's experience and not intended to be general claims as to product performance.



AMS
Henry Mummy Wrap™

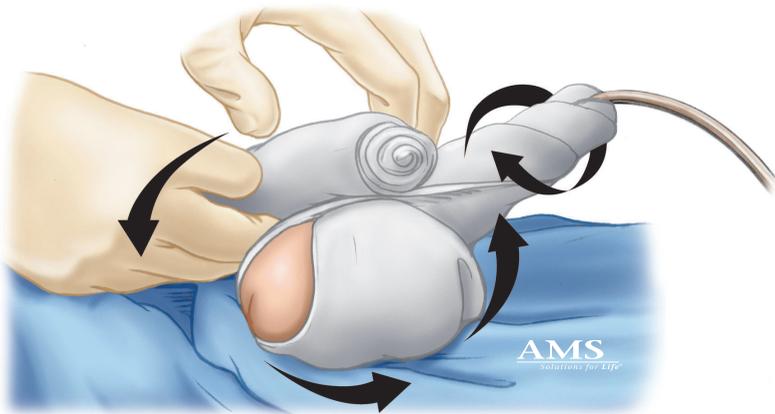


HENRY MUMMY WRAP™

Designed for use with the AMS 700MS™ Series.

INSTRUCTIONS:

- ▲ Loosely wrap penis and scrotum with a 4 inch roll dressing
- ▲ Be sure that both testicles and the pump are incorporated completely up into the wrap
- ▲ Using porous tape, cover penis and scrotum
- ▲ For complete instructions, review attached CD

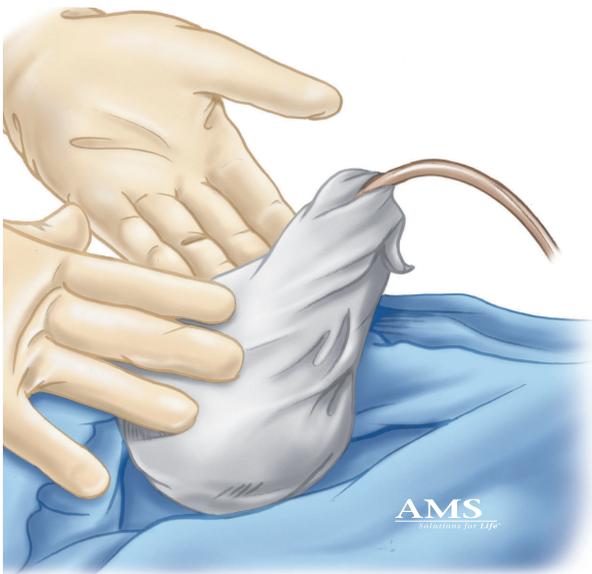

AMS
Solutions for Life™

American Medical Systems, Inc.
World Headquarters
10700 Bren Road West
Minnetonka, MN 55343
USA
Phone: 952 930 6000
Fax: 952 930 6157
www.AmericanMedicalSystems.com

American Medical Systems
Europe B.V.
Straatweg 66H
3621 BR Breukelen
THE NETHERLANDS
Phone: 31 346 258 100
Fax: 31 346 258 130

American Medical Systems
Australia Pty. Ltd.
Suite 2, Level 2
460 Church Street
North Parramatta 2150
NSW AUSTRALIA
Phone: 61 2 8838 8800
Fax: 61 2 9890 9899

American Medical Systems
Canada Inc.
P.O. Box 461
Guelph, Ontario
N1H6K9 CANADA
Phone: 519 826 5333
Fax: 519 821 1356